


Upgrading system software Shockmaster 300

1. Installation of software to USB-stick

① Visit our Library
(<http://partners.gymna-uniphy.com/>)


- ↳ Login
 - ↳ Shockwave Therapy
 - ↳ Technical support
 - ↳ ShockMaster 300
 - ↳ Software ShockMaster 300

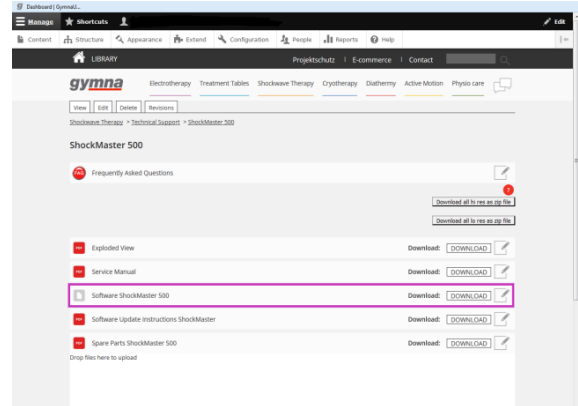


② Depending on the browser, you will get a message to confirm if you want to run or save this file. Choose to save the file.




③ Extract the file with a file archiver like 7-Zip. Insert **an empty USB stick** and choose to extract the file by selecting the browse button.

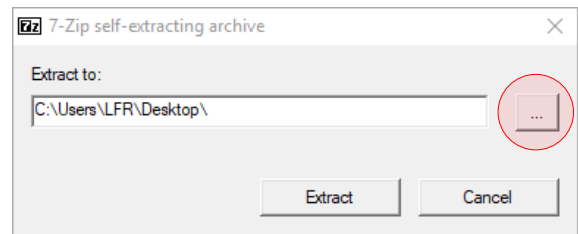

④ Select the drive where the empty USB stick is installed.

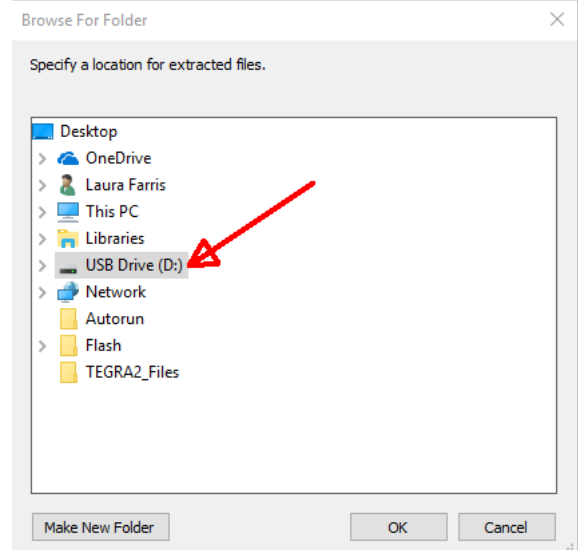


- 5 Now click on the Extract button.

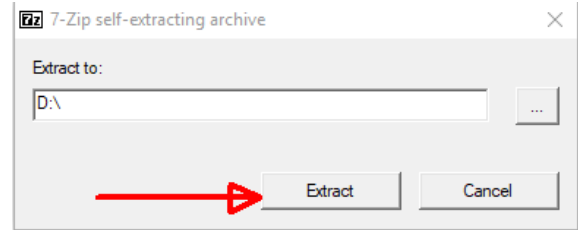

- 6 When exploring the USB content you now should see 3 files

- Autorun
- Flash
- TEGRA2_Files

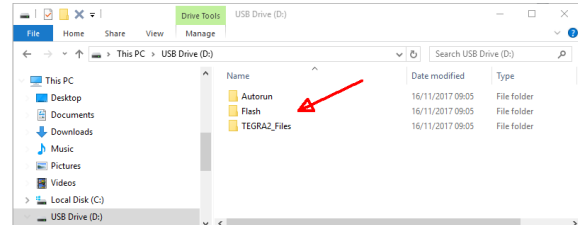

- 7 The USB stick is now ready to perform the upgrade on the Shockmaster 300.

2. Device software update

- 1 Insert the memory stick with the software update after the device has been switched on.




- 2 After a few seconds a popup will appear.

Carefully follow the instructions on the screen.




3. Software version check

- 1 Once the update is finished, check if the version is correct. Push the settings button.

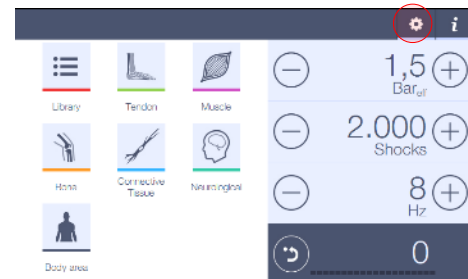


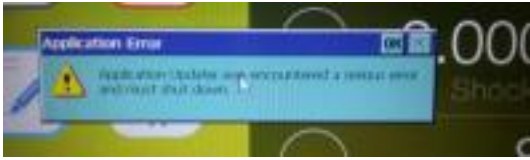
2

Select the info menu. You will find the software version on the right side of the screen: Software version x.x.x (e.g. 5.6.6).

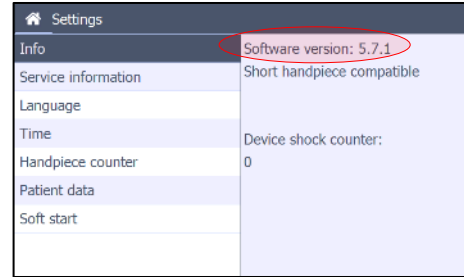

4. Troubleshooting

| | PROBLEM DESCRIPTION | DIAGNOSE AID | | | | | | | | | | | | | | | | | | | | |
|---------------------------------------|-------------------------------------|--|------|------|---------------------------------------|---|------|------|------|------|---------|-----------------|-------------|--|-------|-----------------|-------------|--|--------------|-----------------|-------------|--|
| <p>1</p> | <p>The updater is not starting.</p> | <ul style="list-style-type: none"> Verify the software is properly installed on the memory-stick. If you only see the installer "exe" file on the memory-stick, <div data-bbox="878 821 1409 951" style="border: 1px solid gray; padding: 5px; margin: 5px 0;"> <table border="1"> <thead> <tr> <th>Name</th> <th>Size</th> </tr> </thead> <tbody> <tr> <td> ELOE007-SW200-V602 Shockmaster300.exe</td> <td>8</td> </tr> </tbody> </table> </div> <p>you did wrong. Clear the memory-stick & "RUN" the installer to UNPACK the files on the memory-stick. (see §6.2 Installing the USB-stick) After succesfull unpacking, the content of the USB-stick should look like this:</p> <div data-bbox="878 1171 1409 1339" style="border: 1px solid gray; padding: 5px; margin: 5px 0;"> <table border="1"> <thead> <tr> <th>Name</th> <th>Date</th> <th>Type</th> <th>Size</th> </tr> </thead> <tbody> <tr> <td> Autorun</td> <td>8/06/2018 15:14</td> <td>File folder</td> <td></td> </tr> <tr> <td> Flash</td> <td>8/06/2018 15:14</td> <td>File folder</td> <td></td> </tr> <tr> <td> TEGRA2_Files</td> <td>8/06/2018 15:14</td> <td>File folder</td> <td></td> </tr> </tbody> </table> </div> Verify the memory stick is FAT or FAT32 formatted. (formating the memory-stick is the quickest and easiest way to ensure the memory-stick is empty) | Name | Size | ELOE007-SW200-V602 Shockmaster300.exe | 8 | Name | Date | Type | Size | Autorun | 8/06/2018 15:14 | File folder | | Flash | 8/06/2018 15:14 | File folder | | TEGRA2_Files | 8/06/2018 15:14 | File folder | |
| Name | Size | | | | | | | | | | | | | | | | | | | | | |
| ELOE007-SW200-V602 Shockmaster300.exe | 8 | | | | | | | | | | | | | | | | | | | | | |
| Name | Date | Type | Size | | | | | | | | | | | | | | | | | | | |
| Autorun | 8/06/2018 15:14 | File folder | | | | | | | | | | | | | | | | | | | | |
| Flash | 8/06/2018 15:14 | File folder | | | | | | | | | | | | | | | | | | | | |
| TEGRA2_Files | 8/06/2018 15:14 | File folder | | | | | | | | | | | | | | | | | | | | |

| | | |
|---|---|--|
| | |  <p>in case the memory stick is NTFS formatted, change this to FAT or FAT32. NTFS will NOT work.</p> <ul style="list-style-type: none"> • Do not use Memory sticks with a pre-installed OS. • Insure the USB wiring inside the main unit is conneted to the User interface board |
| 2 | Screen remains black after the update | <ul style="list-style-type: none"> • Make sure you updated to the latest software version. (v6.0.3 or later) • When still not working try upgrading in 2 steps first upgrade to v5.0.2 then install the newest version |
| 3 | <p>If you get the following error message: "Application Updater.exe encountered a serious error and must shut down"</p>  <p>or a "Missing files" error message</p> | <ul style="list-style-type: none"> • The files were not correctly written on the memory stick >> re-extract the files on an "EMPTY" memory stick. Maybe try another memory stick. Make sure the file extracting & copying procedure is finished. Don't just pull out the memory stick from it's port, but use the "Safely Remove Hardware" option from Windows. |
| 4 | If none of the above helps | <ul style="list-style-type: none"> • Contact the service department |